

Intelligent design

V8 Nexus and Platinum 5 are closely integrated to provide intelligent, assistive features to streamline the analytical and reporting process.

1 State-of-the-art Quality Control package

Platinum 5's quality control features include proactive alerting for failed or expired QC, comprehensive Levey-Jennings charting and statistical reporting tools to track quality control performance, and the convenient input of control data simply by scanning your assay sheet bar code.

2 Intelligent, user-friendly interpretation tools


Platinum 5 makes interpretation and diagnosis quicker, easier and more reliable with a range of sophisticated viewing and editing capabilities, including our innovative Second Derivative tool which allows the operator to see small peaks or other features which could otherwise be missed.

3 Automatic data retrieval and comparison

Perform detailed comparison of Serum Protein results alongside Immunodisplacements or historical patient data. Platinum 5 allows you to compare any live or archived data, to get a true picture of the patient's progression. Search across databases for practically unlimited capacity.


Instant status and control of your V8 Nexus

Platinum 5 combines all management facilities into a single, at-a-glance overview of your analyser. Smart automated start-up and shut-down together with proactive maintenance procedures mean that your system will be ready and waiting when you arrive, and geared for maximum efficiency throughout the day.

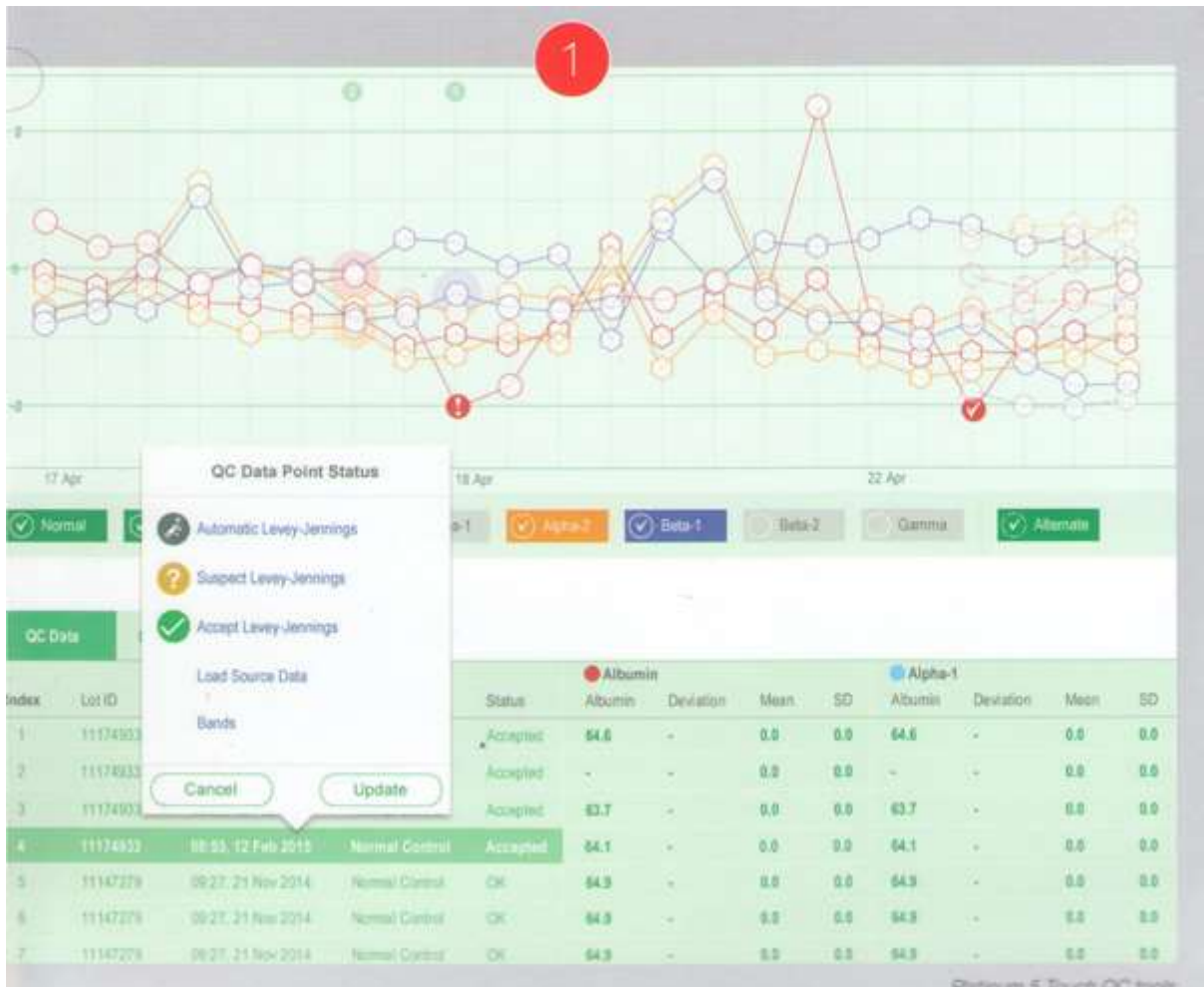

Colour-coded result status previews


		SPE:spe
	Ashley Thompson 	22348576 SPE:spe
	Peter Burnley 	56740578 SPE:spe
	Eilleen Johns 	11089713 SPE:spe
	Arch Stanton 	64245722 SPE:spe
	Alastair Stewart 	90421005 SPE:spe
	Mary Timms-Fewster	22348576

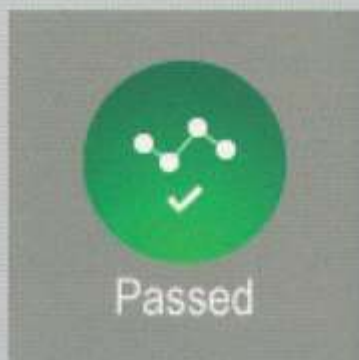

QC status indicator

	Arthur Burnell 	85457010 SPE:spe
	Margaret Hindle 	56740578 SPE:spe
	Scott Middleton 	11264814 SPE:spe
	Richard Procter 	13533984 SPE:spe
	Ashley Thompson 	22348576 SPE:spe

Instant analysis progress

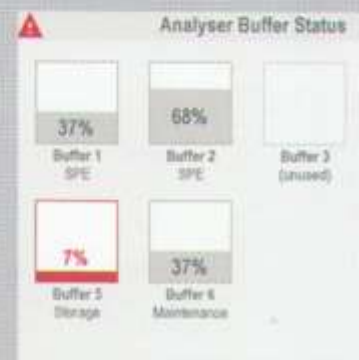

Consumable levels

Ordering information

V8 Nexus 8-channel analyser



V8 Nexus is available as a standard configuration supporting all existing V8 tests, or you can upgrade your Nexus capillary system to support the Fast SP test with the Fast SP Upgrade Kit (800010).

Cat. No.	Description
800008	V8 V8 Nexus (220V) including Platinum 5 Standard Edition, LCD monitor and PC
800018	V8 V8 Nexus (110V) including Platinum 5 Standard Edition, LCD monitor and PC
800010	V8 Fast Serum Protein Upgrade Kit allowing 96 SP tests per hour

Platinum 5 analytical software



Platinum 5 is 64-bit Windows software, available either in basic form or supporting the new touchscreen-compatible user interface.

Cat. No.	Description
400500	Platinum Software — Platinum 5 Standard Edition Basic Platinum interface
400501	Platinum Software — Platinum 5 Touch Edition Requires Platinum 5 Standard Edition, touchscreen and 64-bit Windows 10 PC
400502	Platinum Software — Platinum 5 Plus Pack Expert System, Networking and FlexLab connectivity (requires Platinum 5)
400591	High-Definition LCD Touchscreen Requires Platinum 5 Standard Edition and 64-bit Windows 10 PC
400592	64-bit Windows 10 PC
400593	Transparency Scanner

SAS semi-automated gel analyser systems

Extend the V8 Nexus workflow with Helena Biosciences' gel electrophoresis method portfolio, including low and high throughput Immunofixation and Haemoglobin tests.



1531: SAS-1 Plus



1212: SAS-2



1527: SAS-3



1529: SAS-4

Suitable for low-to-medium throughputs

Suitable for medium-to-high throughputs